

EXPERIENCE THE DIFFERENCE



Lightweight, IPX7 rated*, limited on-board image storage capability** and ergonomic detector design are just a few of the beneficial new features.

* An IPX7 rating does not indicate compliance with, and the detectors have not been tested for compliance with, any other IPX_ ratings.

** When operated in Standalone Mode.

THE NEW SERIES OF CANON CXDI DIGITAL DETECTORS

- CXDI-710C Wireless Digital Radiography System
- CXDI-810C Wireless Digital Radiography System

VIRTUAL IMAGING
A CANON COMPANY

Canon

Lightweight AED* Wireless Detectors

Utilizing strong carbon fiber, this new series of Canon detectors are among the lightest weight detectors currently available. Despite being lightweight, the carbon fiber chassis and frame provide high performance and high durability, tested for the rigors of demanding daily use. The detectors offer superb quality and reliability that you have come to expect from Canon.

- CXDI-710CW Detector: 13.8 x 16.8 in 5.07 lbs w/ battery
- CXDI-810CW Detector: 13.8 x 10.8 in 3.97 lbs w/ battery

* Automatic Exposure Detection



Flexible Charging Capabilities

Each CXDI-10 Series Detector includes a new, multi-functional docking station and a battery charger.

The Docking Station enables:

- Detector check-in
- Battery charging while in the Detector
- Transfer of images from detector taken in Standalone Mode
- Rapid charging

Battery Charger:


- Charge up to (2) batteries
- Compatible with CXDI-01 Series and CXDI-10 Series detector batteries
- Equipped with indicator lights

IPX7 Rated*

Each detector is IPX7 rated for submersion in water up to 1 meter for 30 minutes.*

*An IPX7 rating does not indicate compliance with, and the detectors have not been tested for compliance with, any other IPX_ ratings.

CXDI Control Software NE Version 2.16

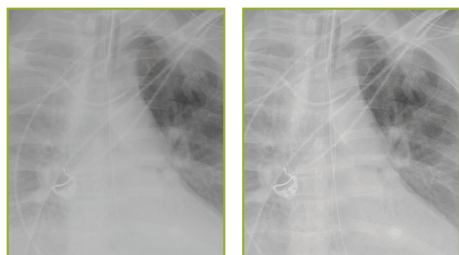

Imaging without Scatter Correction Feature

Imaging with Scatter Correction Feature and without grid

All images are simulated.

CXDI Control Software NE is made exclusively for use with Canon Digital Radiography Systems and helps to optimize workflow and reduce steps needed to complete exams.

- Auto stitching-up to 4 images
- Windows® 7, Windows® 8.1, and Windows® 10 Supported
- Improved noise reduction capability over earlier CXDI models
- IHE and HIPAA Compliant
- DICOM® Structured Dose Reporting
- Support Windows Defender, McAfee VirusScan Enterprise + AntiSpyware Enterprise 8.8, and ESET Endpoint Anti-Virus 6.3.
- Scatter Correction Feature for non-grid exams (sold separately)

Sleek New Detector Design

The sleek, tough and ergonomically sculpted new CXDI-10 Series Wireless Detector design includes the following features to facilitate the user and patient experience:

- Comfortable to hold and easy to grip due to the lightweight and ergonomic handgrips sculpted into the detector on all 4 sides.
- High quality composite materials, low weight and designed with form and function in mind.
- Easy to position and comfortable for patients and technologists due to smooth rounded corners.

Improve Workflow



Optional Status Indicator



Ready Button

Using The 'Ready' Function

When using multiple detectors in one room, a specific detector can be selected not only from the DR modality workstation but also simply by pressing the 'Ready' button directly on the detector or on the optional Status Indicator (sold separately).

On-Board Image Storage

The new Canon CXDI series wireless detectors are equipped with limited, temporary on-board image storage capability when operated in Standalone Mode. Restrictions and limitations apply. For questions and further information call 866-288-9729.



CXDI-710C / CXDI-810C Wireless Specifications¹

Model name:	CXDI-710C Wireless	CXDI-810C Wireless
Purpose:	General Radiography	
Scintillator	Cesium Iodide	
Weight (incl. battery):	5.06 lbs	3.97 lbs
Effective imaging area:	13.8 x 16.8 in	13.8 x 10.8 in
External dimensions:	15.1 x 18.1 x 0.6 in	15.1 x 12.0 x 0.6 in
Image matrix size:	2800 x 3408 pixels	2800 x 2192 pixels
Pixel size:	125 µm	
Resolution:	4.0 lp/mm	
Grey scale:	A/D: 16bit	
Preview image time:	approx. 1 second ²	
Cycle Time:	approx. 7 seconds ²	
Water resistance rating:	IPX7 Rated (For submersion in water up to 1m for 30 min.) ³	
Battery performance:	Generator Connection mode	
	Max. 1000 images @ 7 sec. cycle, Avg. 140 images @ 100 sec. cycle.	
	Non-Generator Connection mode	
	Max. 1000 images @ 7 sec. cycle, Avg. 110 images @ 100 sec. cycle	
Charging performance:	Detector charging in docking station: approx. 120 min. ⁴ In battery charger: approx. 150 min. ⁴	
Wireless channel/band:	2.4 GHz, 5 GHz (W52, W53, W56, W58) ⁵	

¹Specifications subject to change.

²Depending on acquisition mode.

³An IPX7 rating does not indicate compliance with, and the detectors have not been tested for compliance with, any other IPX_x ratings.

⁴At an ambient temperature of 77° F.

⁵W53, W56 supports only in Module receiver mode.

VIRTUAL IMAGING
A CANON COMPANY

Canon

www.usa.canon.com/dr | 866-288-9729
5600 Broken Sound Blvd., Boca Raton, FL 33487

Canon and CXDI are registered trademarks of Canon Inc. in the United States and may also be registered trademarks or trademarks in other countries. McAfee is a trademark of McAfee LLC in the United States and other countries. ESET is a trademark of ESET, spol. s r. o. or ESET. Windows is a registered trademark of Microsoft Corporation in the United States and/or other countries. DICOM is a registered trademark of the National Electrical Manufacturers Association (NEMA), for its standards publications relating to digital communications of medical information. All third-party brand names and product names are registered trademarks or trademarks of their respective owners. Specifications and availability are subject to change without notice. Not responsible for typographic errors.

©2017 Virtual Imaging, Inc., a Canon company. All rights reserved.

MK-0314, Rev. 1